SPECTRAL UV-C

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Using a turbine or handpiece in a dental surgery generates polluting spray that remain suspended for hours.

The maximum concentration of germs settles in a 2 metre range around the dentist's chair, and it is those germs that come into contact with the medical staff, causing a series of problems that can only be partially avoided using masks, goggles and gloves.

A safe, efficient and economic methods needs to be imposed, to prevent professional risks that derive from air contagion.

De Marco proposes SPECTRAL UV-C, which is based on a closed cycle forced ventilation system.

The air that is sucked into the module passes through a mechanical filter positioned on the infeed mouth.

The largest pollutants are held back there, preventing them from depositing on the germicide lamps.

After this, the air moves into direct contact with the mercury-vapour lamps which, thanks to UV-C radiation and a specular screen with a high reflecting capacity, apply the maximum germicide action.

This effectively decontaminates the air coming out of the mouth.

The great advantage of this system is the total lack of danger for those who are present in the room, because not even a minimum amount of UV-C radiation is released by the module.

Continual and constant air disinfection is therefore possible in any environment and during every work phase.

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Spectral UV-C it is also available on a floor base

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